

MRS II[®] Male System

RE-ADJUSTMENT GUIDE

Long-term Re-adjustment

PATIENT MAY NEED SEVERAL RE-ADJUSTMENTS TO REACH DEFINITIVE CONTINENCE.

MANDATORY: PATIENT SHOULD BE RE-ADJUSTED WITHIN 7 DAYS EVERY TIME HE IS INCONTINENT.


DO NOT DELAY ANY RE-ADJUSTMENT (except during the first month after surgery when patient is still healing from surgical trauma).




1– At the office or at Day Surgery OR.




2– Apply local anesthesia at the abdominal varitensor location.

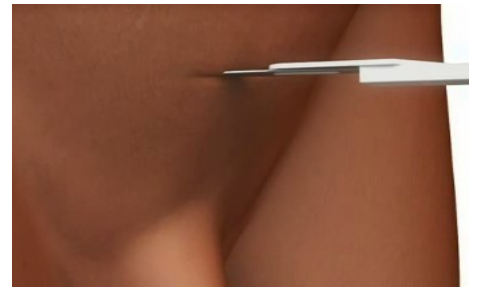

3– Perform an abdominal transversal 3 cm incision and dissect the fat until the varitensor is exposed.

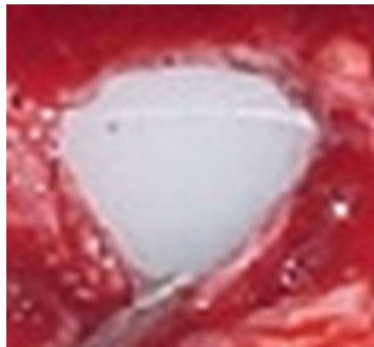


4– Caution: do not cut the traction threads during the varitensor location and exposure.

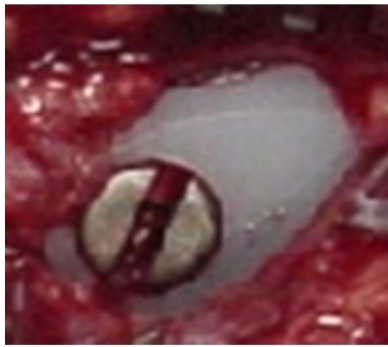


5– Identify the varitensor—manipulator connection side (rotate the varitensor if needed).




6– Open a sterile readjustment kit; Reconnect the manipulator to the varitensor.

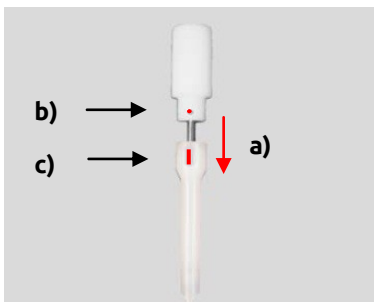

7– Insert the disconnector into the manipulator (a). The disconnector dot mark (b) must be aligned with the manipulator line mark (c). Insert the manipulator tip into the varitensor—manipulator connection groove.

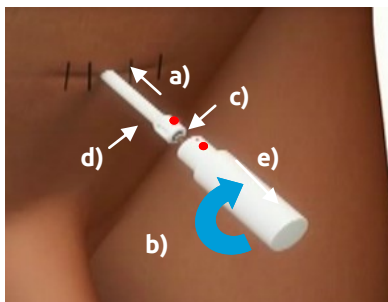

8– While holding the manipulator into the groove (a), turn only the disconnector 1/4 turn (b) until the disconnector dot mark (c) is aligned with the manipulator dot mark (d). Withdraw the disconnector (e). Close the 3 cm. skin incision.

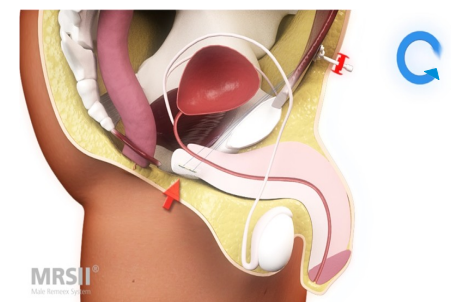

9– Continece Adjustment. As described in post surgical (24-48 hour) adjustment from point three (3).