


# Vaginal Agenesis Remeex Kit


## SURGICAL GUIDE

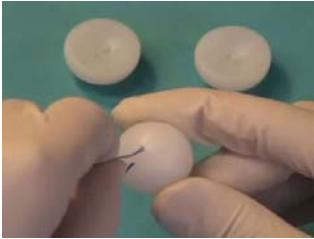


**1.** Pass a suture thread through the traction ball.



**2.** This suture thread will allow the surgeon to extract the traction balls after the neovagina is formed.

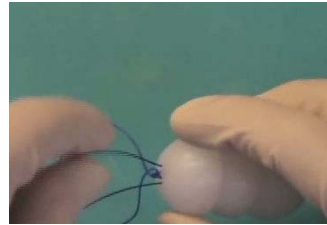

**3.** Pass two traction threads through the traction balls.

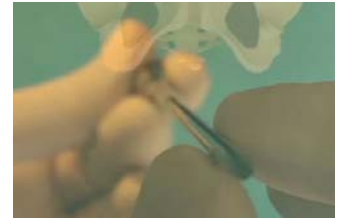

**4.** Insert the vaginal needle through the vaginal membrane to allow surgeon to catch traction threads by laparoscopy.




**5.** Insert the threads through the vaginal needle and pass them through the vaginal membrane.



**6.** Surgeon will catch the traction threads by laparoscopy.




**7.** Rotating the spiral of the sheath, remove the thread of the needle to pull it up by laparoscopy.

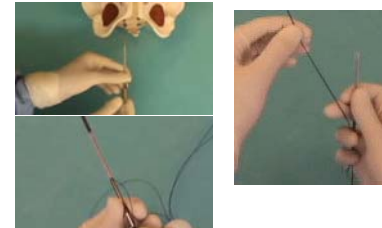


**8.** Repeat the vaginal needle procedure on the opposite side.




**9.** Place the soft support on the abdomen to avoid irritations of the abdominal wall.

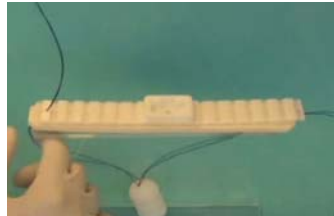

**10.** Place the base plate to give a better support to the varitensor over the patient's abdomen. It also separates the traction threads to have a better traction of the vagina.

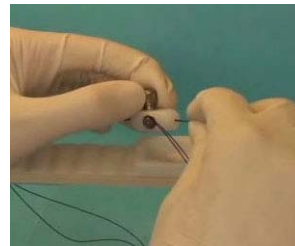


**11.** Insert the threads through the lateral holes of the varitensor. Be sure that you have the same amount of thread in both sides of the varitensor.

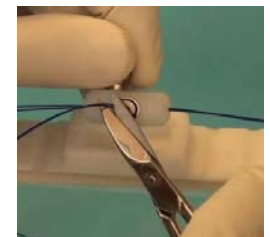

**12.** Close the varitensor with the screwdriver and cut the thread from the center of the varitensor.



**13.** Turn the manipulator clockwise to place the Varitensor on the correct position.

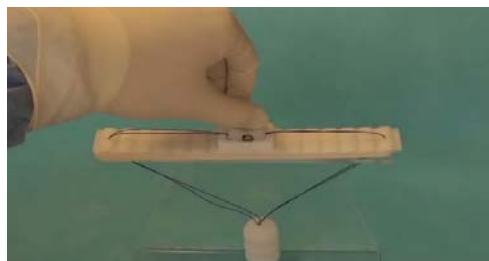

**14.** The following 7-8 days after the intervention patient will adjust AGERS vaginal pressure level.

**15.** Three complete turns morning.  
Three complete turns midday.  
Three complete turns afternoon.