


NEEDLELESS System


The single incision SUI mesh,
a less invasive solution for female SUI.

INSERTION DEVICE FOR NEEDLELESS SYSTEM

IMPROVES AND SOFTENS THE NL INSERTION

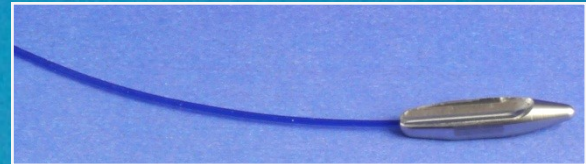


1.- Introduction of the forceps into the NL (*)

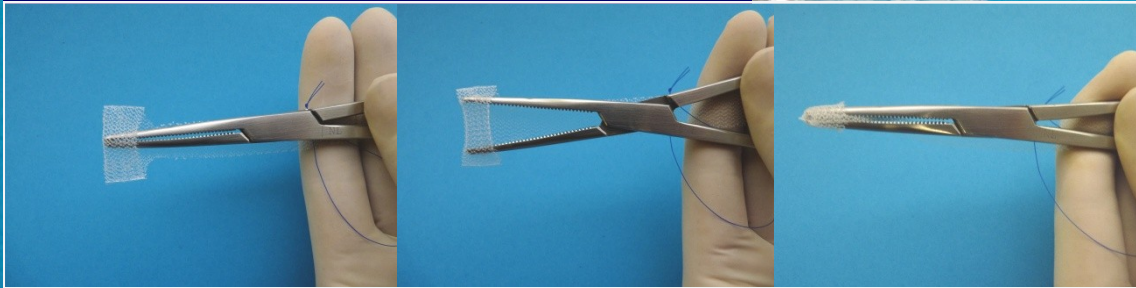

2.- Placement of the NL Insertion Tip for the NL sling insertion



3.- Pull out of the NL Insertion Tip before the forceps withdrawal

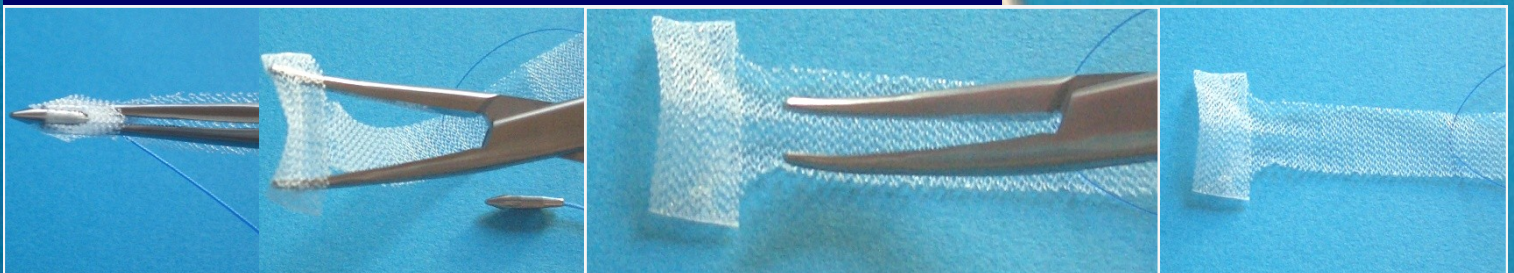

(*) Surgeons can use any universal forceps, but it is recommended to use it with a straight thin tip or a slightly curved thin tip.

www.neomedic.com