
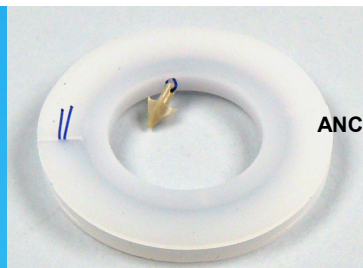



ANCHORSURE® System

SURGICAL GUIDE

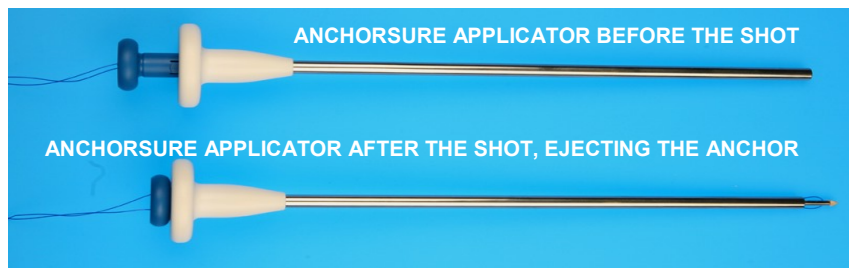


ANCHOR WITH THE THREAD INSIDE ITS PACKAGE


(*) The disposable version of the Anchorsure disposable kit has a BLUE internal shaft handle. The reusable version has a WHITE internal shaft handle (not available in the US market). The external shaft handle is always WHITE for both disposable and reusable versions.

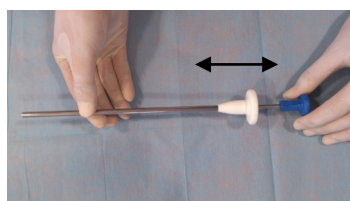


1. Extract the internal shaft from the Anchorsure applicator.

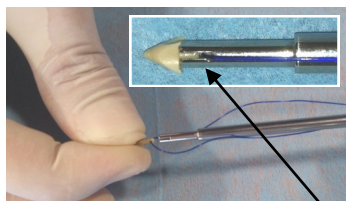


2. Insert the Anchor inside the tip of the internal shaft. Threads must fit inside the bottom slots.

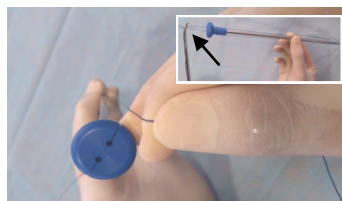


3. Place the threads through the slots of the internal handle, clamp the threads with a mosquito.

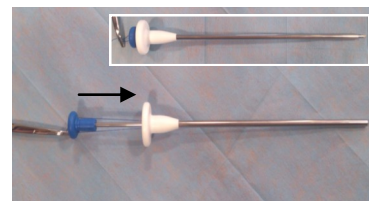


4. Insert the internal shaft inside the external shaft. Hide the anchor pressing the tip of the applicator against a rigid surface.

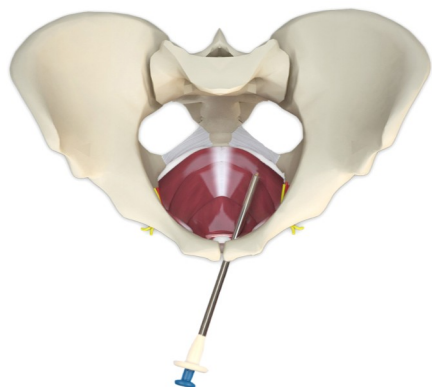


5. Insert the applicator inside the vaginal incision to reach the sacrospinous ligament.

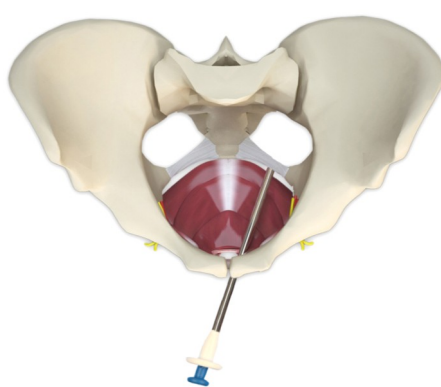


6. Place the system tip compressing the WHITE handle (external shaft) against the Sacrospinous ligament.

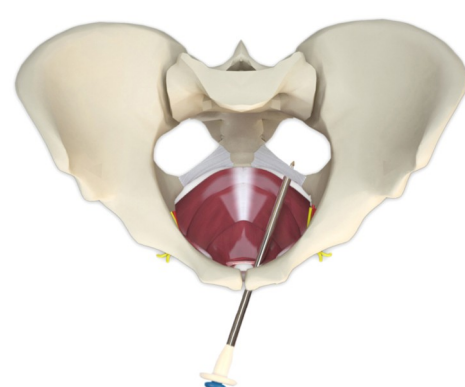


7. Maintaining the WHITE handle firmly compressed against the Sacrospinous ligament, push the internal handle down to the end.

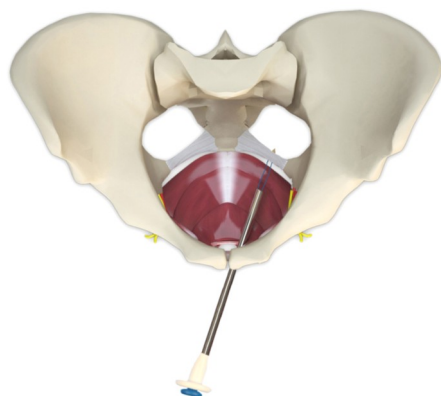

8. Unclamp the threads.

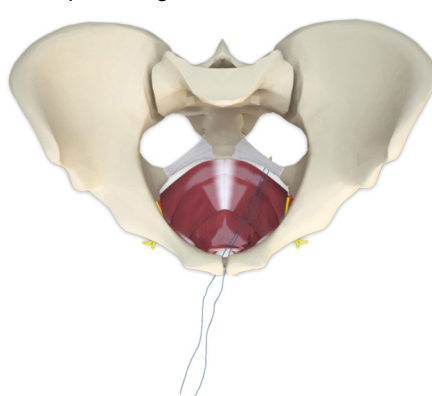

9. Pull out the applicator.

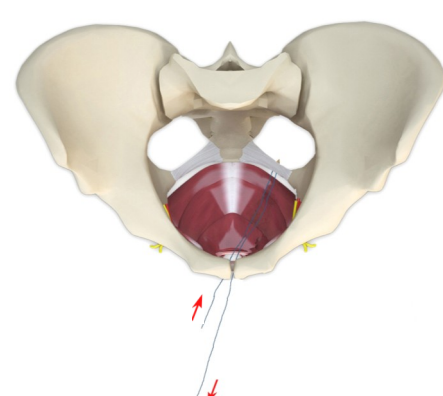

10. With the pulley system, thread is free for any required movement. (**)

11. Check the fixation of the anchor by firmly pulling the threads towards you.

(**) Avoid overpulling completely only one thread, as it will disengage the thread from the anchor.